



**ROTHBAND**

RADIATION PROTECTION

# **INSTRUCTIONS FOR USE**

MULTI USE TABLE SCREEN

VERSION 2.0

**UK  
CA CE**

## Table of Contents

<b>INTRODUCTION .....</b>	<b>3</b>
<b>INTENDED USE.....</b>	<b>3</b>
<b>REGULATORY COMPLIANCE.....</b>	<b>3</b>
<b>CAUTIONS AND WARNINGS.....</b>	<b>4</b>
<b>COMPONENTS .....</b>	<b>5</b>
<b>CLEANING INSTRUCTIONS.....</b>	<b>5</b>
Lead Curtain .....	5
Metal Framework .....	5
<b>SPECIFICATIONS.....</b>	<b>5</b>
<b>WARRANTY .....</b>	<b>6</b>
<b>CONTACT DETAILS .....</b>	<b>6</b>

## INTRODUCTION

The Multi use table Screen is made from flexible lead vinyl sheet with two handles which are design to fit a range of DIN rail sizes. The handles can be positioned in a number of different places along the screen to allow them to be fully configurable to a range of surgical tables. It is available in 0.5mm Pb or 1mm Pb, depending on requirements.

### PRODUCT CODES

TS03

## INTENDED USE

For use as protection against secondary radiation. Only to be used after guidance from a qualified RPA and in compliance with relevant local rules.

## REGULATORY COMPLIANCE

The Wipeable foam positioning pad range complies with MDR (EU) 2017/745 (EU) and MDD 93/42/EEC (UKCA) concerning medical devices (class I) and has been designed and manufactured to comply with the following standards:

Standard	Description
EN ISO 61331-1 2014 BBG*	Protective devices against diagnostic medical X-radiation. Determination of attenuation properties of materials.
EN ISO 14971:2012	Medical devices - Application of risk management to medical devices (ISO 14971:2007, Corrected version 2007-10-01)
EN ISO 15223-1:2016	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements.

## CAUTIONS AND WARNINGS



- Only to be used after guidance from a qualified RPA and in compliance with relevant local rules.
- Should be part of your regular testing programs as per lead aprons.
- Do not use if any obvious tears or cracks are apparent.
- Employ caution when moving over uneven floor surfaces.
- Lower the top screen to its lowest point before transport.
- Do not allow sharp objects to penetrate the lead curtain.
- The above warnings & cautions and any safety considerations should be observed on a routine and regular basis, not only upon installation.


## COMPONENTS



LEAD PANEL



ADJUSTABLE HANDLE

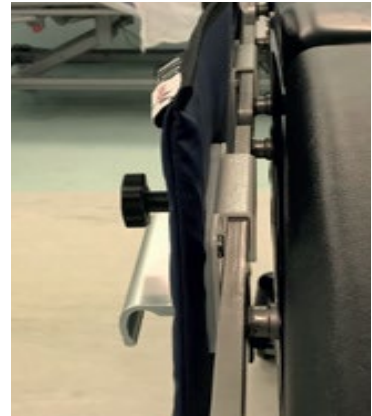

DIN ATTACHMENTS

## CLEANING INSTRUCTIONS


### Lead Curtain

The outer covers may be cleaned using mild disinfectant or detergent. Strongly acidic, alkaline, alcohol/phenol based, or abrasive products should not be used. DO NOT LAUNDER. The product cannot be sterilised.

### Metal Framework

Units can be cleaned with water and normal cleaning solutions. After cleaning, ensure the unit is dry before use.

## SPECIFICATIONS

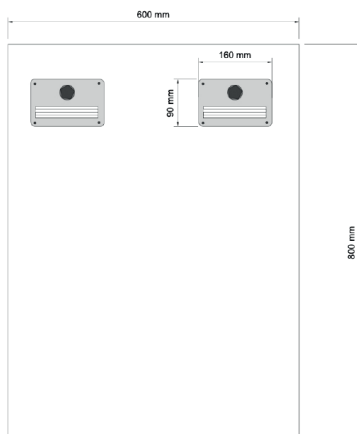

Lead Equivalence mm Pb	0.5, 1.0
Options	Single Panel / 200mm Slats
Dimensions	850mm x 600mm (Other options available)

## WARRANTY

This product comes with a 12 month warranty

## CONTACT DETAILS



WSR Medical Solutions  
(Trading as Rothband)  
Unit E Bridgewater Close,  
Network 65 Business Park,  
Burnley,  
Lancashire,  
BB11 5TE,  
United Kingdom

Tel: 01706 830086

e-mail: [sales@rothband.com](mailto:sales@rothband.com)



Eurolink Europe Compliance Limited, Dublin  
D02 AY86